**Description:**

The Model 7900 CRYOTherm Single-Tank Console provides comfortable and convenient cold hydrotherapy. Water in the tank is precisely maintained at selected temperatures with a robust refrigeration system and electronic temperature control.

Specifications:

Weight / Tank Empty	1,850 Pounds
Weight / Tank Full	4,754 Pounds
Tank Size	72" L x 60" W x 34" D
Tank Volume	46.6 Cubic Feet = 348 Gallons
Tank Surface Area	3840 Square Inches = 26.7 Square feet
Tank Perimeter	264 Inches = 22 Feet
Tank Construction	12 Gauge, Type 304 Stainless Steel
Frame Size	126" L x 72" W x 39.5" H (w/Leveling Feet 40" - 41")
Pit Size	167" L x 74" W x 41" D (see site plan for utility locations)
Water Temperature Range	40° F to 70° F
Refrigeration System	2-1/4 Ton Water Cooled
Full Load Amperage	27 Amps
Circuit Breaker	2 Pole, 40 Amp GFCI
Cold Water Supply Connection	Pipe Size = 3/4" Copper PSI = 20 Min. 80 Max.
Gravity Drain Outlet	Pipe Size = 1-1/2" Copper Flow = 40 GPM

****Please read thoroughly before use and retain for later reference****



Grimm Scientific Industries, Inc.
P.O. Box 2143, Marietta, OH 45750
www.grimmscientific.com

Tel: 800-223-5395
Fax: 740-374-5745
e-mail: grimm@ee.net

Important Safeguards:

The following safety precautions should always be followed when using CRYOTherm.

1. Use CRYOTherm only as directed in this manual. Improper use can cause injury.
2. Do not plug CRYOTherm into any electrical outlet that is not protected by a ground fault interrupter and matches the voltage and amperage specification.
3. Do not modify the power cord in any way or use with any type extension cord.
4. Do not permit any electrical appliance such as television, radios, lamps, clocks, fans, etc. within 5 feet of the tank water, doing so creates the risk of electrical shock.
5. Maintain CRYOTherm as described in this manual. Improper maintenance can cause injury.
6. Do not modify CRYOTherm in any way or attach additional appliances to the frame without consultation with the Grimm Scientific factory.
7. Do not use CRYOTherm if any broken or non-functional components are found, doing so may result in additional damage.
8. Use only Grimm-approved parts and service, improper parts or service can cause harm to the system and/or users.
9. To prevent the transmission of pathogens, people with infectious diseases or open wounds should not use the CRYOTherm.
10. Thoroughly clean the tanks with a disinfectant at least once a week. Refer to the care and maintenance section for further safeguards and instructions for cleaning CRYOTherm, use of an antibacterial agent when filling the tanks helps prevent or reduce infection.
11. Do not utilize warm or hot water treatment where there is hemorrhaging or swelling.
12. Patients with any medical contraindications to cryotherapy, thermotherapy, or hydrotherapy should not use CRYOTherm. Some conditions may include but are not limited to; obesity, heart disease, low or high BP, circulatory problems, diabetes, sickle cell anemia.
13. Prolonged immersion in the tubs may cause harm.
 - *Be aware that patients may develop heat stress syndrome when CRYOTherm is used at high temperatures for prolonged periods.*
 - *Be aware that patients may develop hypothermia when CRYOTherm is used at low temperatures for prolonged periods.*
14. Never allow patients to use drugs or alcohol before using CRYOTherm, doing so increases the risk of hypothermia, hyperthermia, loss of consciousness, and drowning.
15. Pregnant or possibly pregnant women should consult their physician before using CRYOTherm.


Important Safeguards (continued):

16. Advise patients to refrain from adjusting controls, and never leave patients alone.
17. Thoroughly clean the tanks with a disinfectant at least once a week and particularly after treating open lesions (see "Maintenance" section of this manual).


Water Temperature Terminology:

Very Cold	40° F to 55°F	5° C to 13° C
Cold	55° F to 60°F	13° C to 16° C
Warm	90° F to 98°F	32° C to 37° C
Hot	98° F to 104°F	37° C to 40° C
Very Hot	104° F to 110°F	40° C to 43° C

Control Functions:

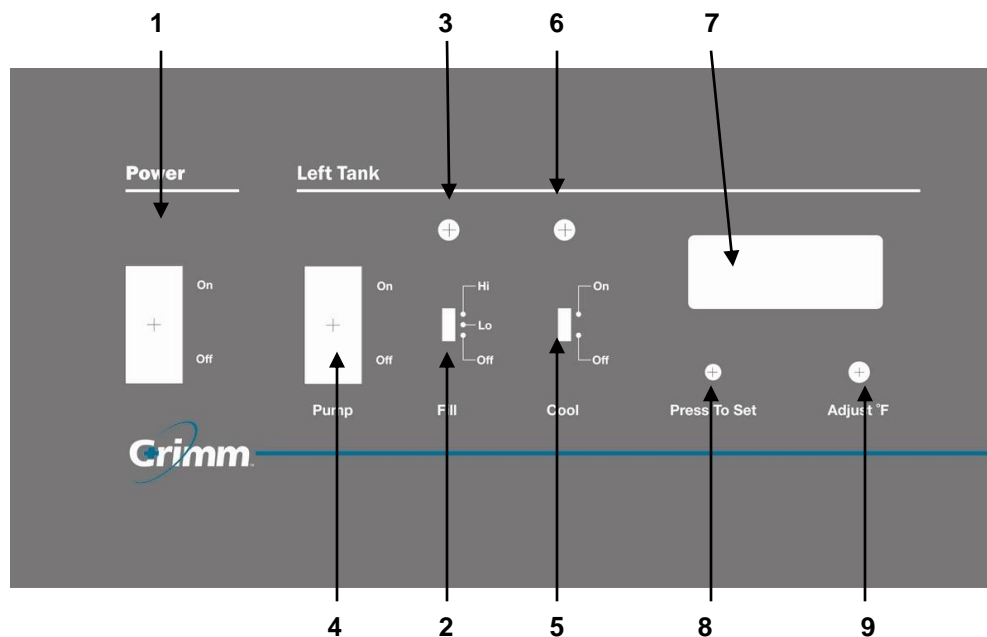

1. **Power Switch** - Activates all electrical components.
2. **Fill Switch** - Activates automatic water level control.
3. **Fill LED** - Indicates when tank is filling.
4. **Pump Switch** - Activates dual speed circulation pump.
5. **Cool Switch** - Activates cooling temperature control.
6. **Cool LED** - Indicates temperature control is asking for cooling.
7. **Digital Display** - Displays water temperature and temperature control setting.
8. **Press-To-Set Button** - Switches digital display from reading the actual water temperature to reading the temperature control setting.
9. **Adjust Slot** - Adjusts (use small screwdriver supplied) temperature control settings when Press-To-Set button is pressed.

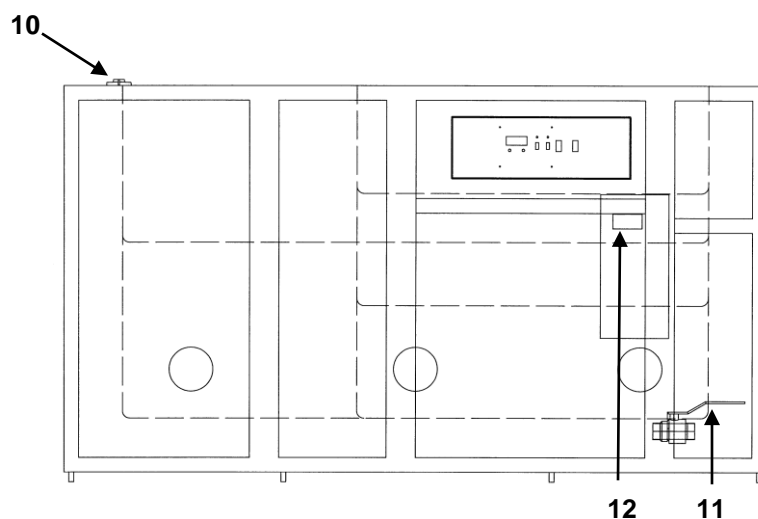

10. **Air Control** - Controls air supply to hydrotherapy jets while pumps are running.
11. **Drain Valve** - Manually drains tank. Power switch should be switched off.
12. **Reset Button** - Press and hold 3 seconds to reset condensing unit controller

Installation:**1. Site Preparation**

The attached Spec Sheet indicates the utilities necessary for installation of the CRYOTherm. Consideration should also be made for the 4,754-pound load this unit will place on the floor and building structure.

2. Level CRYOTherm

Several 5/8-11 stainless steel hex head leveling feet are provided to give uniform support. Thread these into the frame before positioning the unit. Place a level on the top Corian surface and level the CRYOTherm.

3. Drain Connections

A 1-1/2-inch copper drain connection is provided. This should be positioned over an open, trapped floor drain with an adequate air gap. Refer to local plumbing codes for specific requirements.

4. Cold Water Supply Connection

A 3/4" NPT connection is provided for connection to the cold-water supply. An internal backflow preventer protects the supply and requires annual inspection. This supply fills the tank and cools the condensing unit.

When the supply pressure fails, the high-pressure cutout on the refrigeration system will trip making it necessary to press the reset (see #13 under control functions) after the pressure is restored.

5. Electrical Connection

Power requirements for the Model 7900 CRYOTherm are 208 volts, Single-Phase, 60 hertz, 27 amps.

Only plug the CRYOTherm into a voltage-matched NEMA 14-50R 2-pole, 4-wire grounding outlet that is properly grounded, polarized, and 40 Amp ground-fault-circuit-interrupter (GFCI) protected.

Use professional electrical assistance and follow the National Electrical and local codes as a minimum.

Operation:

Operators should have a working knowledge of clinical hydrotherapy and cryotherapy procedures. In addition, each operator should thoroughly read and understand the procedures in this manual before treating patients with CRYOTherm.

1. Fill Tank

The tank must be filled for proper operation of CRYOTherm. To fill the tank first, make sure all control panel switches are in the off position and the drain valve is closed. Press the power switch to on and then slide the fill switches to the "Hi" or "Lo" level and allow the water level to rise. Automatic level switches located behind the overflow grille will control water levels at "Lo" = 4" and "Hi" = 2" to the top of the tank.

Note: *These level controls have a ten-second delay to avoid valve chatter.*

Helpful Hint: *During contrast treatments level switches, can be turned off to avoid loss of temperature due to tanks being constantly filled with water to replace that discharged from the overflow.*

2. Water Supply Requirements

CRYOTherm requires connection to a constant water source in order maintain water level and to allow the refrigeration system to work properly. If water supply is lost the electronic unit controller will shut down the refrigeration system and will require reset to restore function. (See Control Function #13 and the troubleshooting section).

3. Switch Pump "On"

When the water level covers all jets the pump switch (see Control Function #4) may be switched "on" to begin circulating the water.

WARNING: *Do not run the pump when the tank is empty.*

With the dual speed pump switched on water will circulate through the hydrotherapy jets at low speed. The pump may be switched to high speed by depressing the white button on the console top (see Control Function #10). After 20 minutes the pump will automatically resume low speed operation. The air control valve next to the pump speed button can be turned to adjust the amount of aeration during treatment (see Control Function #11).

Important: *The pump must be left on at all times for proper operation. Reliable aeration only occurs at high speed.*

4. Select Tank Temperature

To set the temperature of the tank press and hold the "Press-To-Set" button (see Control Function #8), when depressed the digital display will switch from reading the actual tank temperature to indicating the set point temperature. Insert the small screwdriver that was supplied into the adjacent slot labeled "Adjust °F" (see Control Function #9) and adjust the set point to the desired temperature. When the button is released, the digital display will return to reading the actual tank temperature.

5. Cool Switch

Slide the Cool Switch to "On" and the CRYOTherm will seek the set point temperature and control at that point. Cooling rate is approximately four degrees per hour.

6. Patient Loading & Unloading

Users may access the tank via the compartment steps or over the front and side edges around the perimeter of the tank. Users also have access to the pump speed button and air control dial during the course of treatment however they should never touch the control panel settings beneath the clear plastic cover. CRYOTherm is designed to accommodate all size athletes and support them standing, seated, or kneeling in the tank or sitting anywhere on the exterior Corian surface.

Operation (continued):

During contrast treatments, it is advisable to turn the fill switches of the tank off so that displaced water is not immediately replaced which allows maintenance of more stable temperature for the duration of the treatment. Be sure to turn fill switch back on after treatment is completed.

Important Note:

Water level controls prevent overflow during filling and operation of CRYOTherm, however, patient loading can result in overflow of the tank if entry into the tank is too rapid. Athletes should be instructed to enter the tank slowly, allowing the overflow drain to accommodate the displaced water and prevent overflow of water onto the floor and electrical components.

When exiting CRYOTherm via the compartment steps, care should be taken not to splash excessive water onto the surface of the steps or around the control panel. The clear plastic cover should remain in place whenever CRYOTherm is in use, to avoid splashing of the controls and accidental moving of the switches.

Tripping of the GFCI breaker is possible if a CRYOTherm tank is overflowed, or excessive water is splashed in the area of electrical components. If this occurs the area should be allowed to dry out, and the breaker reset to restore normal operation.

Care & Maintenance:

CRYOTherm is constructed of the highest quality and most durable materials of any hydrotherapy device on the market. Proper care and maintenance of your CRYOTherm will extend the life of its components and provide safe and sanitary environment for your athletes.

1. The entire CRYOTherm should be completely drained, inspected, cleaned, and disinfected at least once a week.
2. CRYOTherm has seamless stainless steel tanks with coved corners which are easily wiped down and don't harbor bacteria. Stains in the tank that do not wipe away easily may be removed using cleaner and a **Maroon Pad**. Apply light to moderate pressure in the direction of the grain of the steel. Never use stainless steel cleaner to clean the tanks.
3. The solid surface Corian exterior surface of CRYOTherm is also seamless, non-porous, and stain resistant. Any stubborn marks that do not easily wipe away can be buffed with a wet **Maroon Pad**, apply light to moderate pressure in a circular pattern.
4. Filter cartridges should be cleaned frequently and replaced regularly to assure they remain effective in removing potentially harmful particles from the system that can damage CRYOTherm components. Consistent use of an **enzyme cleaner** will extend filter life and other hydrotherapy components by enhancing the cleanliness of the piping system.
5. Routine culturing of the tanks should be done. Culturing kits are available from Grimm Scientific. Further guidelines for culturing can be obtained through clinical laboratory or hospital infection control departments.
6. Consult your Grimm representative with questions or problems.

Care & Maintenance (continued):**Weekly Cleaning Procedure**

1. Move all switches on the control panel to the "off" position.
2. Open the drain valve(s) to drain the tanks.
3. When tanks are empty remove the filter cartridge(s). See the **Changing the Filter Cartridge** section of this manual for further instructions.
4. Remove any debris from around the pump suction, drain covers, filter housing, and hydrotherapy jets.
5. Use **pure disinfectant spray** (or comparable product) to clean all exposed surfaces of the tank, the Corian exterior, and the inside of the filter housing. Allow disinfectant to stand on the surfaces for at least 30 seconds before wiping.

Note: *longer contact times may be required for certain pathogens, refer to product label for specific instructions.*

6. Examine tank walls, exterior surfaces, and associated components for damage and visual cleanliness. Remove any stains using the techniques described in the **Care and Maintenance** section of this manual. Report any damage to your Grimm representative.
7. Install a clean filter cartridge(s), refer to the **Changing the Filter Cartridge** section of this manual
8. Close drain valves.
9. Fill tanks according to procedure described in the **Set Up and Operation** section of this manual.
10. Refer to the **Water Balance** section of this manual for information and advice about keeping water clean and sanitized.

Care & Maintenance (continued):

Changing the Filter Cartridge

CRYOTherm is equipped with easily accessible top load filters for quick and convenient cartridge replacement. Replacement cartridges can be purchased from Grimm via our website or phone contact.

1. With the pump, off and the tanks drained, lift the gray tab on the lock ring assembly and turn the ring in a counter clockwise direction to loosen and remove the ring.
2. With the ring removed, grasp the handle on the filter lid and lift, removing the lid from the filter housing.
3. Lift and remove the filter cartridge and clean away any debris remaining inside the filter housing.
4. To clean and disinfect the filter housing, spray the inside surface of the housing and lid with **pure disinfectant spray** (or comparable product) and allow surfaces to remain wet for at least 30 seconds before wiping and reassembling filter.
5. To reassemble, insert a new or cleaned filter cartridge into the housing and replace the lid. Push the lid down completely to assure that it is fully inserted into the housing.

Important: the *rubber o-ring on the lid should be routinely inspected to evaluate wear and need for replacement.*

6. Reinstall the locking ring and turn in a clockwise direction to tighten ring. Do not over tighten.
7. The cartridge that was removed should be discarded or cleaned for future use
8. Proper cleaning of the cartridge includes soaking in filter cleaner, rinsing thoroughly, allowing to dry, and storing in a dry location for future use.

Water Quality Fundamentals:

The three foundational domains of water quality are filtration, water balance, and sanitation. Normal use and proper care of CRYOTherm as described in **Weekly Cleaning Procedure** section of this manual is usually effective in maintaining a balanced and sanitary condition of water without the need for chemical treatment. Certain factors may however affect the condition of the water to the point that some chemical treatment may be needed. We recommend periodic testing of the water to evaluate water quality. The following is a simple guide to testing and, if necessary, treating the tank water in order to assure CRYOTherm is safe and comfortable for your athletes.

Filtration

Mechanical filtration of water removes particles from the water that can make it cloudy and discolored; it does not remove bacteria or viruses. 100% circulation through replaceable cartridge filters keeps CRYOTherm's water clear and free of debris. The cartridges need to be cleaned regularly and replaced periodically to assure they are working properly.

Water Balance

Water balance refers to a state of water in which the proper amount of dissolved material exists in the water to make it an optimal environment for hydrotherapy. Balanced water creates an environment that is comfortable for the user, naturally sanitized, and also protects the components of the CRYOTherm from damage. Water with too much dissolved material can result in deposits forming in the CRYOTherm system, called scaling, while water with too little or no dissolved material can result in corrosion of the surfaces and components. Water balance is more a factor of preserving equipment than a health issue for the user.

CRYOTherm water balance is affected by organics that accumulate over time. Commonly referred to as bio loading, these organics come from body oils, perspiration, proteins, wastes, cosmetics, lotions, soaps, and other sources that are introduced to CRYOTherm water by users and the surrounding environment. Signs of increased organics include foaming, cloudy water, development of a water line, odors, and accumulation of debris in the system.

Source water in certain locales may also exhibit relatively high levels of dissolved metals, minerals, and other contaminants resulting in unbalanced and discolored water which may stain or damage the tank and other components.

The factors that can be measured to evaluate water balance are as follows; the ideal range for each is also indicated:

- pH Ideal range is 7.4 to 7.6 (acceptable 7.2 to 7.8)
- Total alkalinity Ideal range is 80 to 120 ppm
- Calcium hardness Ideal range is 200 to 400 ppm

These factors are easily measured using commercially available test strips and easily adjusted to ideal range using commercially available chemical additives. If applicable, see the **water testing** and **water treatment** section.

Sanitation

Sanitation refers to the process of destroying microorganisms that might cause human disease. Sanitation, also known as disinfection, results in water that is free of bacteria, algae, and other nuisance organisms that affect water quality and pose a health risk to the user. Water temperature, pH, environmental wastes, and user contamination are all factors that can affect sanitation. CRYOTherm is designed to limit, or even remove, the need for chemical treatment of water to achieve sanitation. That being said, we recommend periodic culturing of the water to confirm that no harmful pathogens exist.

Some CRYOTherm owners routinely use commercially available sanitizers to assure the cleanliness of water in the system. There are a number of ways this can be done but in each instance, be sure to strictly follow the manufacturer's instructions for use, storage, and disposal of the chemicals being used. If applicable, see the **water treatment** section for more information and suggestions regarding the use of sanitizers in CRYOTherm.

Water Quality Fundamentals (continued):

Water Testing

The factors that determine water balance are easily measured using test strips that are dipped into the water and then compared to color coded scales to evaluate levels. Characteristics of the source water, amount of use, cleanliness of users, cleaning schedule, temperature of water, and other factors can affect water balance. The most important parameter for CRYOTherm users to monitor and adjust is pH. Signs that you may have a pH or other water balance problem are as follows:

Signs of high pH, total alkalinity, and/or calcium hardness

- Cloudy water
- Accumulation of scale
- Reduced circulation from clogged filters, heaters, or piping
- Eye/skin irritation

Signs of low pH, total alkalinity, and/or calcium hardness

- Etching or staining of surfaces
- Corrosion
- Foaming water
- Eye/skin irritation

Test kits are also available to evaluate the amount of various mineral and metal contaminants in the source water. Your local pool store may also provide this service for you. Based on findings, the water can be treated to remove the damaging materials.

If the decision is made to treat the water with a disinfectant (chlorine or bromine), it will be important to test the water to evaluate the levels of residual disinfectant to make sure it is effectively sanitizing the water without causing irritation of the skin or eyes of the users. Test strips are the easiest way to test for the disinfectant levels in the water. It is important to strictly follow manufacturer's instructions when testing the water and when adding chemicals to the water. Ideal ranges are as follows:

- | | | |
|-------|---------------|---|
| - FC | Free Chlorine | 2 to 4 ppm ideal range, 1 to 4 ppm acceptable range |
| - TBr | Total Bromine | 3 to 5 ppm ideal range |

Water Quality Fundamentals (continued):

Water Treatment

The first step to any chemical treatment of water is to determine if any change in water chemistry is necessary. Because CRYOTherm's water is continuously refreshed with normal use and completely replaced at least weekly by the user, you may find that chemical treatment of the water is not warranted. If routine testing shows values that are not within acceptable ranges, chemical adjustment may be needed.

We have found that pH is the most important factor. When pH is in ideal range, all other factors tend to fall in line. If you are having trouble maintaining an ideal pH, you may first need to adjust Total Alkalinity which serves as a buffer to resist changes to pH levels.

Follow these guidelines after filling the CRYOTherm:

- If there is a known water source problem the water should be treated first pursuant to an established protocol to remove the potentially harmful contaminants.
 - o Your Grimm representative can assist you with establishing proper protocol.
- If Total Alkalinity (TA) is not in ideal range adjust it first.
 - o **SpaPure Alkalinity Increaser**
- Adjust pH once TA is within ideal range and stabilized
 - o High pH – adjust with **SpaPure pH Down**
 - o Low pH – adjust with **SpaPure pH Up**
- If using a sanitizer, add sanitizer only after TA and pH are within ideal ranges and stable, this will assure it works most effectively and reduce the amount you need to use.
- Treat water with **ORB-3 Spa Enzyme No Foam** approximately 30 minutes after introducing sanitizer.

NOTE: When using chemicals always read the labels and strictly follow the manufacturer's instructions regarding dosage, handling, storage, and disposal. All of the products needed to comply with these recommendations can be purchased directly from Grimm Scientific.

Sanitizers

Sanitizers that are part of the chemical family known as halogens are the most commonly utilized method of water disinfection. These products kill microorganisms, including bacteria, by attacking their cell walls through oxidation. They destroy the enzymes and structures inside these cells, which renders them harmless. Chlorine and Bromine are the most widely used sanitizers and come in various forms.

- Chlorine is probably the most recognized water treatment. Typically added to water in a granular form, it is an effective sanitizer for all applications. It is important to maintain chlorine concentration to within acceptable ranges.
- Bromine is another effective sanitizer; it is preferred by some because it is less likely to produce odors and skin irritation. Bromine is considered the best sanitizer for warm water applications since it does not dissipate rapidly. Using bromine tablets in a floating dispenser is a simple and effective way

Factors to consider when choosing the best disinfectant to use in your CRYOTherm and the best method of application are:

- Water temperature
- Usage load
- Source water chemistry
- Chemical storage and safety concerns
- Supervision and maintenance concerns

Water Quality Fundamentals (continued):


Water Testing and Treatment Checklist

Note: ALWAYS carefully read and strictly follow all manufacturer's instructions and recommendations when storing, handling, and adding any chemical product to your CRYOTherm water.

1. After each scheduled weekly cleaning fill tank(s) with water as described in the **Set Up and Operation** section.
2. With pump(s) turned off, use a test strip according to manufacturer's instructions to evaluate the **Total Alkalinity (TA)** and **pH** of the water.
3. If **pH** is out of range (**7.2 – 7.8**) adjust it using **SpaPure pH Up** or **pH Down**.
If **TA** is out of range adjust it first using **SpaPure Alkalinity Increaser**.
4. If using a sanitizer add it now and turn the pump(s) on.
SpaPure Bromine Tablets in feeder
5. **Note:** Floating dispensers should be removed while the tank is in use
6. 30 minutes after introducing the sanitizer treat the water with the appropriate amount of **ORB-3 Spa Enzyme No Foam**.
7. Recheck **pH** and **sanitizer** levels periodically throughout the week to assure the water is staying in range and adjust as necessary. After a few weeks, you will establish a workable routine.
8. If, after 3-4 days of operation, the water shows signs of imbalance, treat water with **SpaPure Oxidizing Shock** and add additional **ORB-3 Spa Enzyme No Foam**. **
9. **Note:** If shock and enzyme product does not bring the water into balance consider draining and cleaning the tank.
10. Contact your Grimm consultant with questions, problems or any specialized needs.

Service:**1. Trouble Shooting**

Problem	Possible Cause
Nothing happens when Power switch is switched "On".	Power cord unplugged. GFCI breaker tripped.
CRYOTherm won't cool.	Check that the water supply is on, then press and hold (for 3 seconds) the start button on the Electronic Unit Controller shown below. Check that pump is on and pumping water
Cloudy Water	Check filter cartridge.
Water circulation impaired.	Check pump suction fittings for debris. Check filter cartridge.

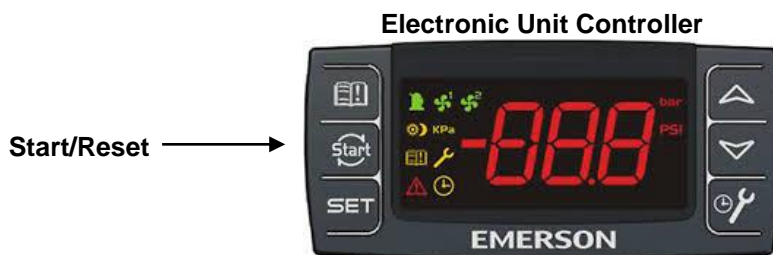**2. One Year Warranty**

The CRYOTherm warranty applies to purchaser and any subsequent owner. Grimm Scientific Industries, Inc. (warrantor) warrants this product to be free from defects in material and workmanship under normal use for one year from the date the product is shipped. The warrantor may opt to replace the product, rather than repair it. This warranty does not cover damage resulting from accidents, improper use, or maintenance of this product, and in no case shall warrantor's liability exceed the original purchase price paid for the product. This warranty is expressly in lieu of all other express warranties.

3. Repairs

Should repairs be needed on your CRYOTherm, simply call the factory and ask to speak with a service technician. We will repair it free during the first year and quote/invoice a reasonable charge for parts and labor provided after the warranty expires.

4. Inspection & Calibration

Every year a qualified service technician should check the CRYOTherm. Temperature control calibration should be verified with a certified thermometer and electrical system tested for grounding and leakage current. Grimm can provide this service.

5. Call 1-800-223-5395

All Grimm products are sold and serviced directly by the factory. Call us toll-free Monday through Friday, 8:00 a.m. to 5: p.m. EDT about your thermal therapy needs. We are often available before and after hours so call us anytime.